




Exclusively for Kinesio® Professionals

KINESIO® TEX GOLD™

EDF PRE-CUT™

HOW TO APPLY:

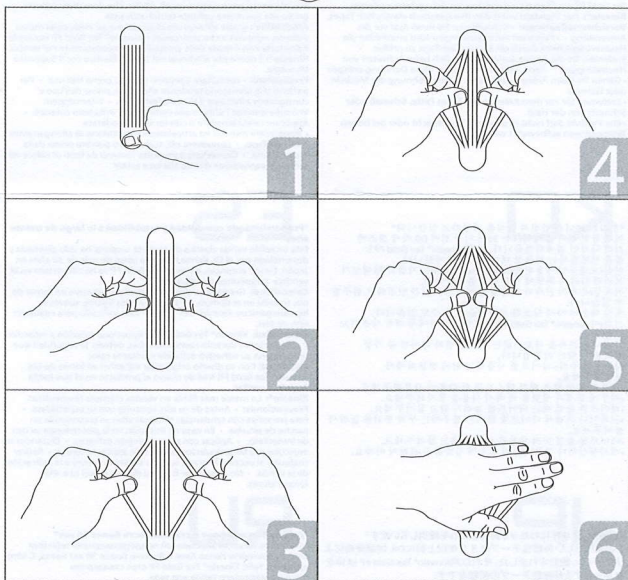

1. Position the ends of the web cut strip gently at either end of target tissue. Do not yet rub into place.

2. Gently grasp center of outer tail with fingertips or tweezers. Try to avoid excessively touching the adhesive.

3. Spread both outside tails sideways simultaneously as far as possible from center of application without creating too much tension.

4. Repeat with succeeding tails, working from the outside in, creating a similar width of space in the slit between tails, until reaching the center.

5. The center slit should end up about the same width as the others.

6. Rub surface of tape gently to activate the adhesive.



HYPO-ALLERGENIC



LATEX-FREE



MULTIPLE DAY WEAR



WATER-RESISTANT



100% COTTON



FINGERPRINT TECHNOLOGY

KINESIO HOLDING CORPORATION
4001 Masthead St. SE
Albuquerque, NM 87119
855.488.TAPE(8273)
www.KinesioTape.com



EDF-IFU-001-RevA

Phone: +31.70.345.8570 Fax: +31.70.346.7299

PATENTED JP1433989
US D678,685 S

EMERGO EUROPE
Monsieur St.
2518BH The Hague,
The Netherlands

